QBC™ Venous Blood Tubes

4245-200-001(A)
2006/06

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Italiano: pagine 10 - 12
Español: páginas 13 - 15

IMPORTANT : FOR DETAILED INFORMATION ON THE **QBC** METHOD, AND USE, TESTING, AND PERFORMANCE OF VENOUS-BLOOD TUBES, CONSULT THE **QBC** SYSTEM OPERATOR'S MANUAL.

INTENDED USE

For use only with **QBC**-series centrifugal hematology systems yielding: hematocrit; hemoglobin; platelet count; white blood cell count; granulocyte count (% and number); lymphocyte/ monocyte count (% and number).

REAGENTS

Venous tube coatings: Acridine orange, potassium oxalate, agglutinating agent and stabilizers (see package labelling for concentrations).

Warnings and Precautions

For *in vitro* Diagnostic Use.

WARNING: ACRIDINE ORANGE REAGENT MAY BE TOXIC. DO NOT INGEST. AVOID CONTACT WITH SKIN, EYES AND CLOTHING.

Storage: KEEP **QBC** TUBES TIGHTLY STOPPERED IN THEIR OPAQUE GLASS VIAL WHEN NOT IN USE. Remove tubes one by one as needed. Be sure to re-stopper the vial each time. Do NOT remove the strip of desiccant from the vial of tubes. Store tubes in a dark, dry place at 16 ° to 32 °C. Protect from moisture, direct light and heat.

Stability: Unopened vial expiration - printed date on label applies only to tubes in the unopened vial. Opened vial expiration - tubes can be used for 30 days after first opening the vial. Record the opening date in the space provided on the label. Do NOT use **QBC** tubes after 30 days from opening.

SPECIMEN COLLECTION

WARNING : BLOOD SPECIMENS MAY CONTAIN THE HEPATITIS B VIRUS OR HUMAN T-LYMPHOTROPIC VIRUS TYPE III (HTLV-111/LAV). ALWAYS HANDLE BLOOD SPECIMENS AS POTENTIAL BIOHAZARDS.

Venous Blood: Draw venous blood specimens with a **Vacutainer™** Brand Blood Collection tube or other blood drawing device containing EDTA. To assure an acceptable blood-to-anticoagulant ratio, fill the collection tube with blood to at least 2/3 of its total fill volume. Remove the collection needle and destroy after use. Thoroughly mix the specimen with the EDTA anticoagulant. If hemolysis or small clots are observed, discard the specimen.

Anticoagulants: Venous-Blood specimens for **QBC** tests must be anticoagulated only with disodium or tri-potassium EDTA. Do not use other anticoagulants.

TEST PROCEDURES

Materials Provided: **QBC** Venous-Blood Tubes, tube closures and floats, and multi-language package insert.

CAUTION: EXERCISE CARE WHEN HANDLING **QBC** TUBES. THE TUBES ARE MADE OF GLASS AND MAY BREAK

IMPORTANT:

- VENOUS BLOOD SPECIMENS MUST BE CENTRIFUGED WITHIN 90 MIN OF BLOOD DRAW TO OBTAIN THE FULL PANEL OF **QBC** PARAMETERS.
- IF A PLATELET COUNT IS NOT REQUIRED, THE SPECIMEN MAY BE UP TO 4 H OLD BEFORE COMPLETION OF CENTRIFUGATION.
- **QBC** TEST MUST BE PERFORMED AT 20° TO 32 °C (68° TO 90 °F).

Procedure Steps:

1. **FILL AND SEAL THE TUBE:** Insert end of tube nearest red lines into Pipetter. Press tube fully into Pipetter barrel. Depress plunger, then insert tube into specimen of well-mixed anticoagulated blood. Release plunger slowly. Check that blood level is within ± 1 mm of black fill line. Wipe excess blood from surface of tube. Manually twist and push closure onto tube. Be sure closure is on straight and tight. Remove tube from Pipetter.
2. **MIX TUBE AND INSERT FLOAT:** Gently roll tube between fingers AT LEAST 10 TIMES OR FOR AT LEAST 5 SEC TO MIX THE BLOOD WITH TUBE COATINGS. Using forceps supplied with **QBC** Reader, insert float into open end, or slide tube over float. Raise unsealed end of tube slightly above horizontal to prevent float from sliding out. Press end of float against a clean surface until float is inside tube. NOTE: NEVER TOUCH THE FLOAT WITH FINGERS. Promptly proceed to Step 3.
3. **CENTRIFUGE FOR 5 MIN:** Place **QBC** Tube(s) on rotor of centrifuge according to procedures in Operator's Manual. If odd number of tubes are being centrifuged at one time, use empty tube (with closure and float) for balancing. Secure rotor cover and press centrifuge lid down until latched. Centrifuge for 5 min. PROMPTLY REMOVE TUBES FROM THE CENTRIFUGE. Check that plasma level is between two red lines. If above or below lines, discard tube and prepare fresh tube.
 - Avoid picking up or handling tube below red indicator lines or placing tubes on warm surfaces or under intense light.
 - Do not twist or move closure after centrifugation.
 - If not read immediately, store tubes in vertical, closure-down position to maintain cell layer boundaries.

Centrifuged tubes are stable for up to 4 h before reading, provided tubes are stored vertically (closure down) away from heat and direct light.

4. **READ LAYER LENGTHS AND RECORD TEST VALUES :** Place centrifuged tube into **QBC** Reader. Be sure Reader is in "VEN" mode. Read 6 interfaces according to procedures in **QBC** Operator's Manual. Record tests values before proceeding to next test. Note : Since cell interfaces may become slightly blurred from prolonged exposure to heat, read **QBC** blood tubes as soon as possible after inserting in Reader.

SYSTEM CHECK

Perform a daily pre-test check of the **QBC** Reader with the Venous Check Tube supplied with the instrument. Procedures are fully described in the applicable Operator's Manual.

ORDERING INFORMATION

QBC Tubes for Venous Blood : Case of 5 packs - 200 tubes, closures and floats per pack - Reorder n° 424237. Box of 100 tubes - floats and closures - Reorder n° 424245.

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A division of Woodley Equipment Company Ltd